

# Medigene®

## ULTRASOUND GEL

### PRODUCT DESCRIPTION

Medigene Ultrasound Gel (Non Sterile) is for use on intact skin only. Not for sterile procedures, or when procedures are next to non-intact skin (surgical site or wound).

This product has applications in Physical Therapy, Sports Medicine, Radiology and scans for bladder, abdominal, vascular areas and external pregnancy scanning.

Medigene Ultrasound Gel is gentle on the skin and is water-based. It is easily wiped off with a clean paper towel or tissue.

- NON-GREASY
- WATER-BASED
- GENTLE
- READY-TO-USE
- PERFUME FREE
- COLOUR FREE

### INGREDIENTS

- Comprises only non hazardous ingredients.

### STORAGE

- Store in a cool dry place.

### AVAILABLE SIZES

Available in 1L, 5L, 20L & 200L.

### PRODUCT USES

- External pregnancy scanning
- Physical therapy
- Sports medicine
- Radiology
- External bladder, abdominal and vascular scans

### DIRECTIONS FOR USE

- Pour generously onto the area to be scanned.
- When examination is complete, wipe off excess with a clean dry tissue, paper towel or cloth.
- Never try to re-introduce used or contaminated product into the original container.

A non-greasy, water-based lubricant used in ultrasound examinations on intact skin (non-sterile).



### SAFETY INSTRUCTIONS

- For external use only.

#### FIRST AID

- If on skin, wash with soap & water. Rinse well with clean water.
- If in eyes, rinse well with water. Remove any contact lenses. Continue rinsing for 5 minutes.
- If inhaled, move to fresh air & rest where you can breathe comfortably.
- If ingested, do not induce vomiting. Drink a glass of water.

Seek medical attention/advice if you feel unwell or any irritation persists.

POISON INFO CENTRES:  
NEW ZEALAND 0800 764 766  
AUSTRALIA 13 1126

